



## **PetED™ Veterinary Education and Training Resources**

[www.PetED4VetCE.com](http://www.PetED4VetCE.com)

215-990-1769

### **Veterinary Anesthesia Machine Checklist**

- Insure Adequate O2 Supply
- Vaporizer Level
- CO2 Absorbent Fresh?
- Pressure Test
  - Should hold at 20 cm H2O
  - Common places for leaks
    - Breathing Bag
    - Breathing Circuit
    - Unidirectional Breathing Valves
    - CO2 Absorber Gasket
    - Pressure Relief Valve
    - Vaporizer Inlet and Outlet Connections
    - Vaporizer Filler Cap
    - Fresh gas Delivery Hose and Connection

### **Breathing Bag Size Chart**

<b>Patient Weight</b>	<b>Bag Size</b>
15 lbs or less	500 ml
15-30 lbs	1 Liter
30-60 lbs	2 liter
60-100 lbs	3 Liter
100+	5 Liter