



PetED™ Veterinary Education and Training Resources

www.PetED4VetCE.com

215-990-1769

Veterinary Anesthesia Machine Checklist

- Insure Adequate O2 Supply
- Vaporizer Level
- CO2 Absorbent Fresh?
- Pressure Test
 - Should hold at 20 cm H2O
 - Common places for leaks
 - Breathing Bag
 - Breathing Circuit
 - Unidirectional Breathing Valves
 - CO2 Absorber Gasket
 - Pressure Relief Valve
 - Vaporizer Inlet and Outlet Connections
 - Vaporizer Filler Cap
 - Fresh gas Delivery Hose and Connection

Breathing Bag Size Chart

Patient Weight	Bag Size
15 lbs or less	500 ml
15-30 lbs	1 Liter
30-60 lbs	2 liter
60-100 lbs	3 Liter
100+	5 Liter